


# KRATOS POSEIDON with SafetyGrip®

KPSG780

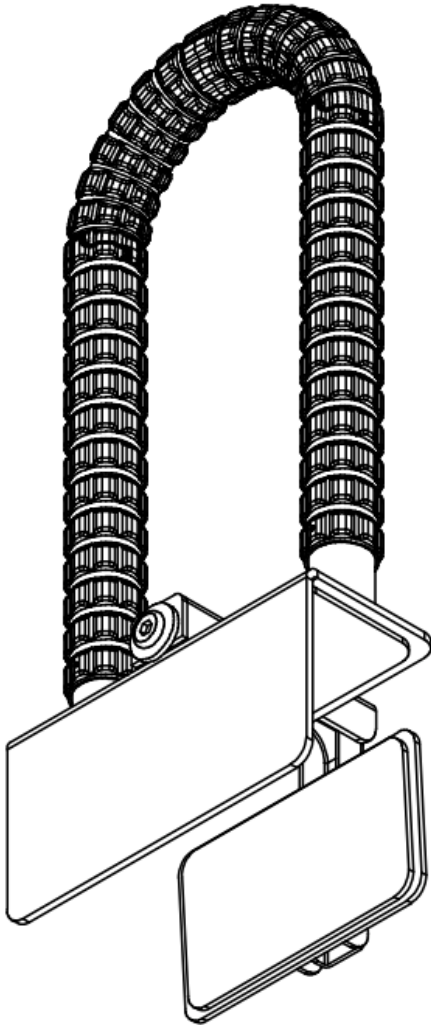


## Safety Information

**!** **IMPORTANT:** If you are unsure that you can install this product correctly, hire a qualified professional.

If you have any questions, please contact Customer Care at [info@SafetyGrip.com](mailto:info@SafetyGrip.com). For warranty details, please visit <https://SafetyGrip.com/customer-care/#warranty>.

# KIT CONTENTS

The Hex Key needed for this installation is included in the kit.

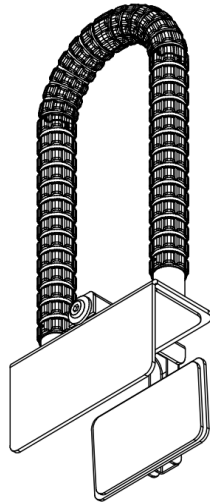

## TOOLS



HEX KEY

# PRODUCT INFORMATION

The KPSG780 Bath Hand Rail comes with our patented Interlocking SafetyGrip®. The KPSG780 is protected with AlphaSan® technology, which provides lasting protection against microbial contamination to all Hartmobility products, delivering unrivaled anti-microbial performance.

- This KPSG780 is designed to assist with balance and stability for persons weighing up to 350lbs (158kg) and is not designed to bear the full weight of the user.
- **NOT DESIGNED FOR FIBERGLASS SURFACES.**

# INSTALLATION

**Installer Note:** Make sure that all surfaces where you want to clamp KPSG780 Bath Hand Rail are clean and dry.


## STEP 1: Position the Bath Hand Rail

Position the Bath Hand Rail over the edge of the bath tub where you want it located.

## STEP 2: Securely Tighten The Clamp

Securely tighten the clamp onto the side of the bath tub using the Hex Key. There are 3 non-slip surfaces on this Bath Hand Rail. Make sure that all 3 are in full contact with your bath tub surfaces where you are mounting this unit. Finally, make sure that the Bath Hand Rail does not move when you push and pull on it.

**Installer Note:** Keep the Hex Key because you will need it to maintain the safety of your Bath Hand rail

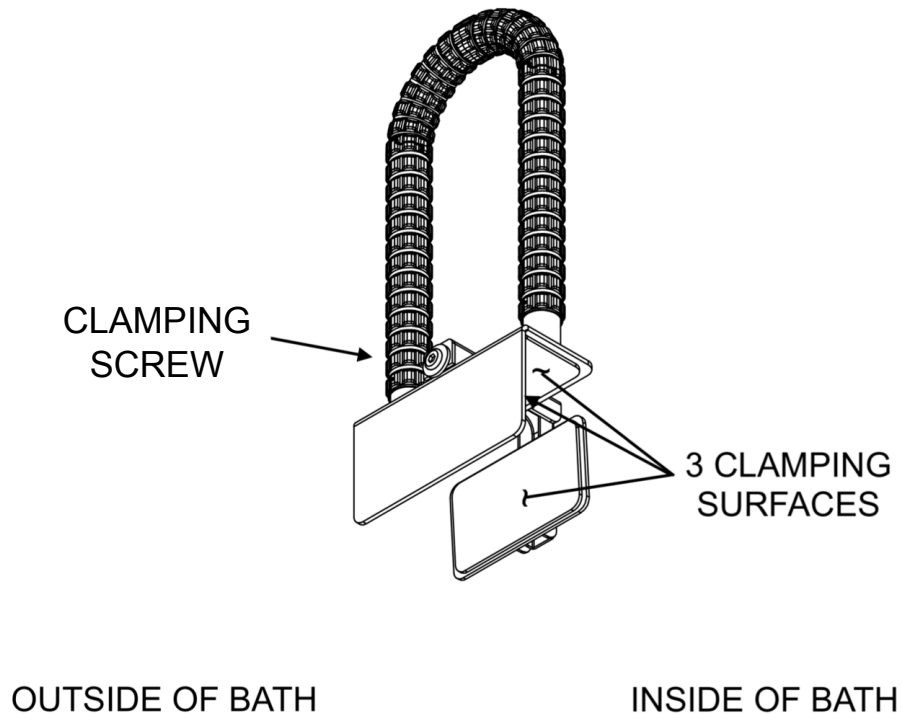

## MAINTENANCE

- The Bath Hand Rail is installed on a smooth surface and can loosen with use. Check the installation and the Clamping Screw weekly.
- To clean your Bath Hand Rail, simply wipe with a clean cloth and any commercially available cleaning solution.

# LIMITED WARRANTY

Hartmobility Inc. product(s) are covered by a limited lifetime parts warranty. This warranty covers against defects in design and workmanship. Hartmobility Inc. will correct any defects in parts or workmanship at no charge.

The warranty does not cover product failure due to installation error, owner misuse, abuse, negligence or normal wear and tear.

For more information on what our Limited Lifetime Warranty covers and does not cover, please visit the Hartmobility Inc. website at [www.Hartmobility.com](http://www.Hartmobility.com) or scan the following QR Code.





**Questions. Comments. Concerns.**

Contact Customer Service today.

[info@SafetyGrip.com](mailto:info@SafetyGrip.com)

[www.SafetyGrip.com](http://www.SafetyGrip.com)

[www.Hartmobility.com](http://www.Hartmobility.com)

1(800) 998-0774

Hartmobility Inc.  
424 Williamson-Zebulon Rd.  
Zebulon, GA 30295