


TITAN POLE ACCESSORY

THSGA605

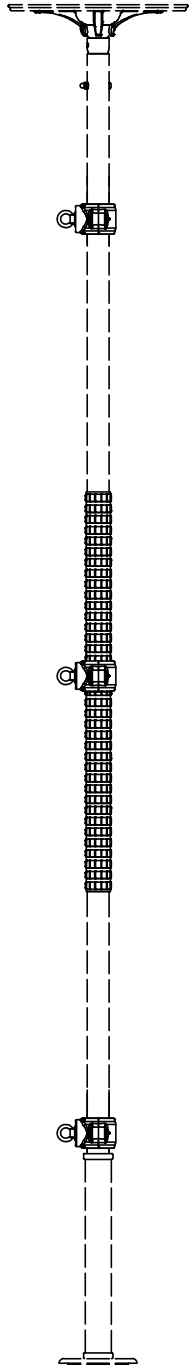

Safety Information




IMPORTANT: If you are unsure that you can install this product correctly, hire a qualified professional.


If you have any questions, please contact Customer Care at info@SafetyGrip.com. For warranty details, please visit <https://SafetyGrip.com/customer-care/#warranty>.

KIT CONTENTS

For regular installations you will need a Step Ladder and Safety Glasses. The Hex Key is included in this kit.

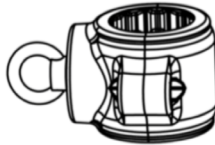

One 582 for
SafetyGrip

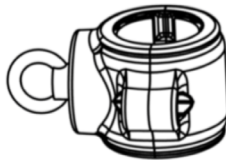

Two 583 Clamps
for Painted Pole



Hex Key

PRODUCT INFORMATION

The THSGA605 parts are protected with AlphaSan® technology, which provides lasting protection against microbial contamination to all Hartmobility products, delivering unrivaled anti-microbial performance.

- THSGA605 comes with our patented Interlocking SafetyGrip®.
- The THSGA605 Exercise Clamps Kit is designed to be used with the TSG610; TSG615 & TSG650 Titan Poles.
- The THSGA605 Exercise Clamps Kit is compatible with all styles of Resistance Band and Resistance Tubing products.

INSTALLATION

STEP 1: Open the clamps


First put on safety glasses. Open the clamps by undoing the screws and nuts using the Hex Key provided.

STEP 2: Place the clamps where you want to use them

Place the clamps where you want to use them, re-insert the screws and nuts and tighten firmly using the Hex Key.

Installer Note 1: There are two clamps - 583 for installation on the smooth painted parts of the Titan pole and one clamp - 582 for installation on the SafetyGrip part of the Titan pole.

Installer Note 2: Should you want the bottom clamp lower, slide the Cover Tube up on the Titan pole, install the clamp lower and slide the Cover Tube back down the Titan pole onto this clamp.

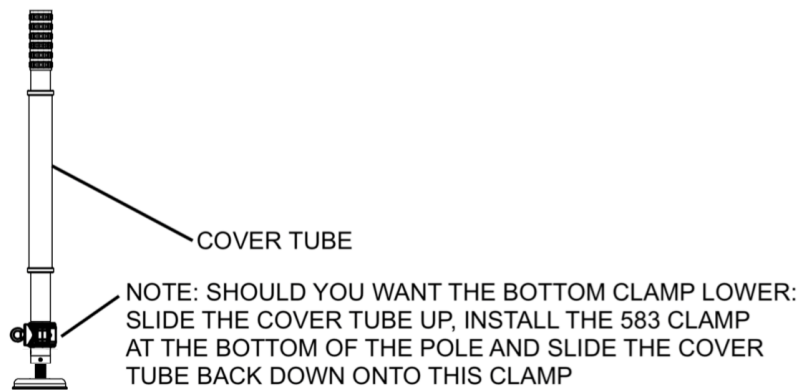

STEP 2

MAINTENANCE

- The clamps could loosen with use. Check them monthly to be sure that they don't loosen

LIMITED WARRANTY

Hartmobility Inc. product(s) are covered by a limited lifetime parts warranty. This warranty covers against defects in design and workmanship. Hartmobility Inc. will correct any defects in parts or workmanship at no charge.

The warranty does not cover product failure due to installation error, owner misuse, abuse, negligence or normal wear and tear.

For more information on what our Limited Lifetime Warranty covers and does not cover, please visit the Hartmobility Inc. website at www.Hartmobility.com or scan the following QR Code.





Questions. Comments. Concerns.

Contact Customer Service today.

info@SafetyGrip.com

www.SafetyGrip.com

www.Hartmobility.com

1(800) 998-0774

Hartmobility Inc.
424 Williamson-Zebulon Rd.
Zebulon, GA 30295