

TITAN POLE ACCESSORY with SafetyGrip®

SG601

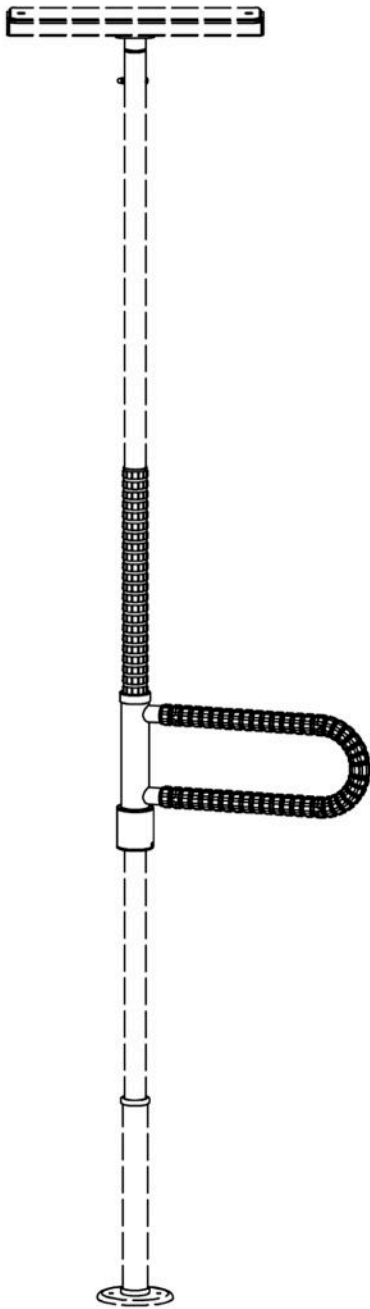



Table of Contents

- SG601 Installation Instruction (5 Pages)

Specifications:

- Product Name: Titan RotoBar
- Product#: SG601
- Maximum User Weight: 350lbs (158kg)

KIT CONTENTS

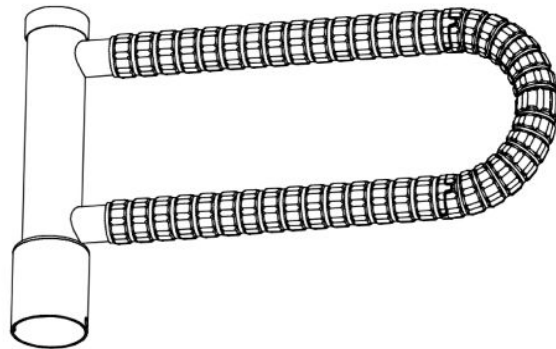

① ROTOBAR



② BOTTOM COGS COLLAR



③ PLASTIC STOP RING



HEX KEY

PRODUCT INFORMATION

IMPORTANT:

◇ When installing the SG601 there is always a risk of entrapment between it and beds, furniture, fixtures and other objects.

◇ If you are installing this SG601 Rotobar on a Titan Pole that is near a bed there must be a gap of at least 15 inches between the Pole and the nearest edge of the bed.

The Titan series feature our patented interlocking SafetyGrip® and the long-lasting protection of embedded antimicrobial AlphaSan® RC5000 silver ions powder for all parts except hardware.

For more details about AlphaSan®: <https://www.milliken.com/en-us/businesses/chemical/product/alphasan>

- The SG601 Safety Grip RotoBar is designed to be used with the SG610; SG615 & SG650 Poles.
- The SG601 is designed to assist with sit-up, reposition, balance, stand, transfer and stability for persons weighing up to 350lbs (158kg) and is **not designed** to bear the full weight of the user.



INSTALLATION


NOTES

1. If you purchased the SG601 RotoBar and the Titan pole together follow the Steps 1 through 4 below.
2. If you already have the Titan pole and it is installed you will need to loosen it and lay it on the floor. Remove the Base and remove the Bottom Cover Tube. Now follow Steps 1 through 4 below.

STEP 1: Slide the Rotating Grab Bar and parts onto the Pole

Slide the RotoBar assembly and the other parts onto the Titan pole from the bottom.

Follow the sequence shown on the Step 1 drawing.

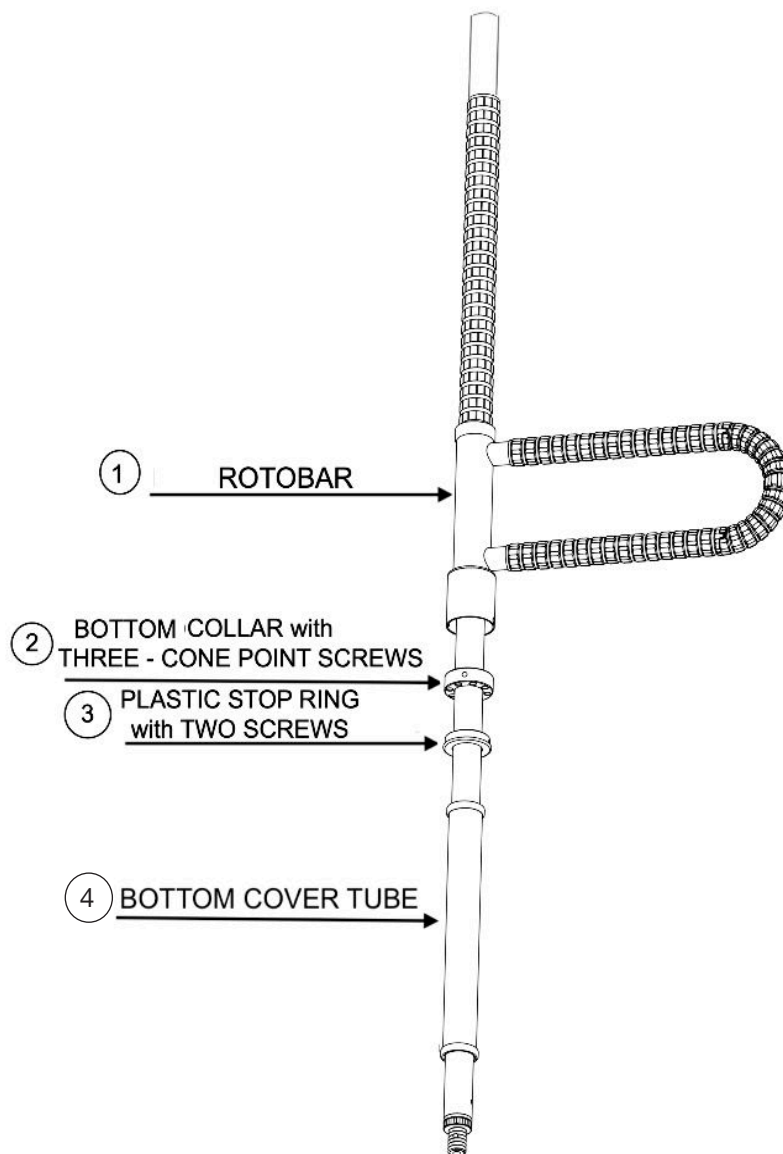

ASSEMBLY of PARTS SEQUENCE
STEP 1

STEP 2: Finish Installing the Pole

Finish installing the Titan pole by following the Pole Assembly Instructions provided with the Titan pole.

STEP 3: Decide how high you want the RotoBar

When the Titan pole is assembled and standing between the floor and ceiling you can decide how high that you want the RotoBar to be from the floor.

Move the Bottom Collar up the Pole to this height.

Installer Note: This Bottom Collar will be the height of the bottom horizontal RotoBar.

Securely tighten the three Security Screws in the Bottom Collar with the Hex Key provided.

STEP 4: Install the Plastic Stop

Install the Plastic Stop Ring into the bottom of the RotoBar with the two screws provided.

Test that the RotoBar lifts and rotates smoothly and drops down and looks securely in place.

MAINTENANCE

- **VERY IMPORTANT:** It is important to inspect the Product(s) frequently.
- Stop using immediately if damaged or broken, or if parts are missing.
- To clean your RotoBar, use a damp cloth with soap and water or mild household cleaner, wipe clean and dry before using.

Warranty

For Warranty details, refer to the Warranty details on the Hartmobility Inc.

website: www.hartmobility.com/Warranty or call 1-800-998-0774 ext. 3



Hartmobility reserves the right to update and revise our products. Refer to our website for the most up-to-date version of this document.





Questions. Comments. Concerns.

Contact Customer Service today.

info@SafetyGrip.com

www.SafetyGrip.com

www.Hartmobility.com

1(800) 998-0774

Hartmobility Inc.
424 Williamson-Zebulon Rd.
Williamson, GA 30292