


# TITAN POLE ACCESSORY

SG605

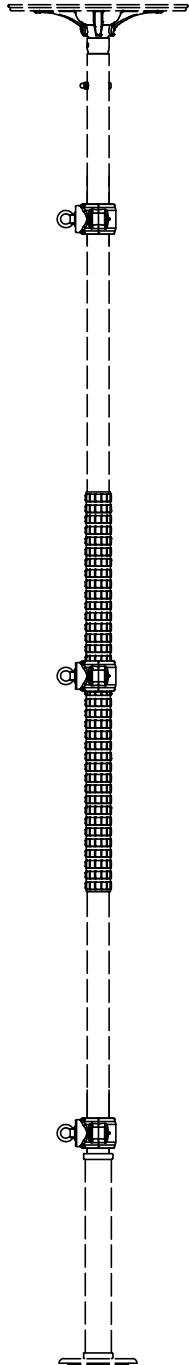

## Table of Contents


- SG605 Installation Instruction (4 Pages)

## Specifications:


- Product Name: Titan Pole Clamps
- Product#: SG605

# KIT CONTENTS

For regular installations you will need a Step Ladder and Safety Glasses. The Hex Key is included in this kit.

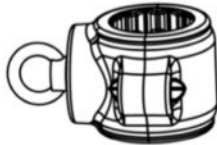

One 582 for  
SafetyGrip

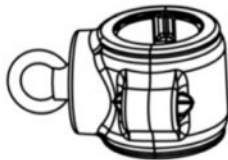

**X 2**

Two 583 Clamps  
for Painted Pole



Hex Key

# PRODUCT INFORMATION

The SG605 feature our patented interlocking SafetyGrip® and the long-lasting protection of embedded antimicrobial AlphaSan® RC5000 silver ions powder for all parts except hardware.

For more details about AlphaSan®: <https://www.milliken.com/en-us/businesses/chemical/product/alphasan>

- The SG605 Titan Pole Clamps is designed to be used with the SG610; SG615 & SG650 Titan Poles.
- The SG605 Titan Pole Clamps is compatible with all styles of Resistance Band and Resistance Tubing products.

# INSTALLATION

## STEP 1: Open the clamps


First put on Safety Glasses. Open the clamps by undoing the screws and nuts using the Hex Key provided.

## STEP 2: Place the clamps where you want to use them

Place the clamps where you want to use them, re-insert the screws and nuts and tighten firmly using the Hex Key.

**Installer Note 1:** There are two clamps - 583 for installation on the smooth painted parts of the Titan pole and one clamp - 582 for installation on the SafetyGrip part of the Titan pole.

**Installer Note 2:** Should you want the bottom clamp lower, slide the Cover Tube up on the Titan pole, install the clamp lower and slide the Cover Tube back down the Titan pole onto this clamp.

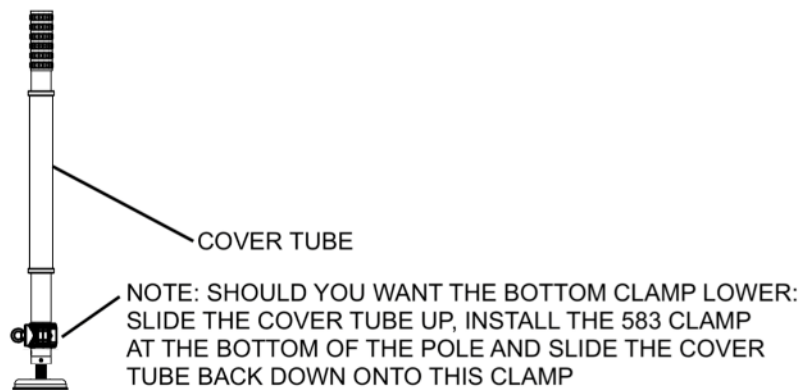

## STEP 2

# MAINTENANCE

- **VERY IMPORTANT:** It is important to inspect the Product(s) frequently.
- Stop using immediately if damaged or broken, or if parts are missing.
- Stop using immediately if Product(s) shift out of the original installed position(s) until they are readjusted into the correct position.

To clean your Titan Pole Clamps, use a damp cloth with soap and water or mild household cleaner, wipe clean and dry before using.



## Warranty

For Warranty details, refer to the Warranty details on the Hartmobility Inc.

website: [www.hartmobility.com/Warranty](http://www.hartmobility.com/Warranty) or call 1-800-998-0774 ext. 3



**Questions. Comments. Concerns.**

Contact Customer Service today.

[info@SafetyGrip.com](mailto:info@SafetyGrip.com)

[www.SafetyGrip.com](http://www.SafetyGrip.com)

[www.Hartmobility.com](http://www.Hartmobility.com)

1(800) 998-0774

Hartmobility Inc.  
424 Williamson-Zebulon Rd.  
Williamson, GA 30292

Hartmobility reserves the right to update and revise our products. Refer to our website for the most up-to-date version of this document.

