

KRATOS SERIES GRAB BARS with SafetyGrip®

MODELS

SG714
(14 in, 35.5 cm)

SG724
(24 in, 61 cm)

SG736
(36 in, 91.5 cm)

SG746
(46 in, 116.8 cm)

ATTENTION!

It is known that all products carry some risks of injury or death. HartMobility Inc has made every effort to minimize these risks and other hazards, but it is impossible to eliminate them all. Therefore, the consumer is responsible for using our products with common sense and taking into account the overall physical and mental health of the user(s). Before installing or using any HartMobility product, read all the accompanying literature and instructions.

WARNING!

INSTALLATION:

- It is important that you carefully read and understand all materials accompanying this product before installation or use. Failure to follow these instructions could result in serious injury to the user(s). Improper installation significantly increases the risk of injury. If you have any questions about the assembly, installation or use of this Product; or any questions about the risks associated with this Product; or any other questions regarding any Hartmobility Inc. Products please contact our Customer Support Help Desk at info@hartmobility.com; or call our Customer Support Help Desk at 1-800-998-0774 ext. 3
- Individuals with certain physical or mental conditions, or those under the influence of drugs or substances that could affect their balance or judgment, are at a higher risk of injury. Such individuals should consult with a physician before using this product.
- Children should not be allowed to play with, climb or hang on this product.
- This product is **not intended** to support the full weight of an individual. It is designed only to provide balance and support while sitting and standing.
- Inspect this product frequently to ensure proper installation.

Agreement Between Installer/User(s) and Hartmobility Inc.

Neither Hartmobility Inc. nor the Seller of this Product will be Liable for improper installation or use of this Product.

Warranty

For Warranty details, refer to the Warranty details on the Hartmobility Inc.

website: www.hartmobility.com/Warranty or call 1-800-998-0774 ext. 3

This work is copyright. Apart from any use permitted under the Copyright Act 1976, no part may be reproduced by any process, nor may any other exclusive right be exercised, without the permission of Hartmobility, Inc. 2023

