

# Warnings and Cautions for ROTOBAR

## ATTENTION!

It is known that all products carry some risks of injury or death. HartMobility Inc has made every effort to minimize these risks and other hazards, but it is impossible to eliminate them all. Therefore, the consumer is responsible for using our products with common sense and taking into account the overall physical and mental health of the user(s). Before installing or using any HartMobility product, read all the accompanying literature and instructions.

## WARNING!

When using this product against or near furniture or fixtures there is a possibility of entrapment. Hartmobility Inc. (Hartmobility) is committed to providing our customers and users with information and education about what Entrapment is and how to avoid Entrapment when using our Products. To assist in understanding the Entrapment Risks, our leaflet called Entrapment Prevention Information is available on our website.

Entrapment risk occurs whenever a user of an APBR can have their head, neck, chest, or other body part trapped in the space between the bed, the mattress or any other object and the APBR. Entrapment can result in serious injury and death.

Before installing or using this Product it is important to consider carefully who will be using it. The F3186-17 ASTM Standard, the CPSC, and the FDA describe users who are particularly at risk of entrapment, suffocation and strangulation as; “*People with Alzheimer’s disease, dementia, or other neurological conditions, or those who are sedated, confused, or frail.*”

If the user has been diagnosed with any of these conditions caregivers, and the installers, must understand and be aware of the Entrapment risks and take full responsibility for the correct installation and use of this Product.

If you are not technically competent to install this Product we recommend that you hire a professional installer/contractor.

### INSTALLATION:

- If you have any questions about the assembly, installation or use of this Product; or any questions about the risks associated with this Product; or any other questions regarding any Hartmobility Inc. Products please contact our Customer Support Help Desk at [info@hartmobility.com](mailto:info@hartmobility.com); or call our Customer Support Help Desk at 1-800-998-0774 ext. 3
- Children should not be allowed to play with, climb or hang on this product.
- This product is **not intended** to support the full weight of an individual. It is designed only to provide balance and support while sitting and standing.
- Inspect this product frequently to ensure proper installation.

### Agreement Between Installer/User(s) and Hartmobility Inc.

Neither Hartmobility Inc. nor the Seller of this Product will be Liable for improper installation or use of this Product.

### Warranty

For Warranty details, refer to the Warranty details on the Hartmobility Inc.

website: [www.hartmobility.com/Warranty](http://www.hartmobility.com/Warranty) or call 1-800-998-0774 ext. 3

This work is copyright. Apart from any use permitted under the Copyright Act 1976, no part may be reproduced by any process, nor may any other exclusive right be exercised, without the permission of Hartmobility, Inc. 2023

