

SafetyGrip® TRAPEZE

SG602

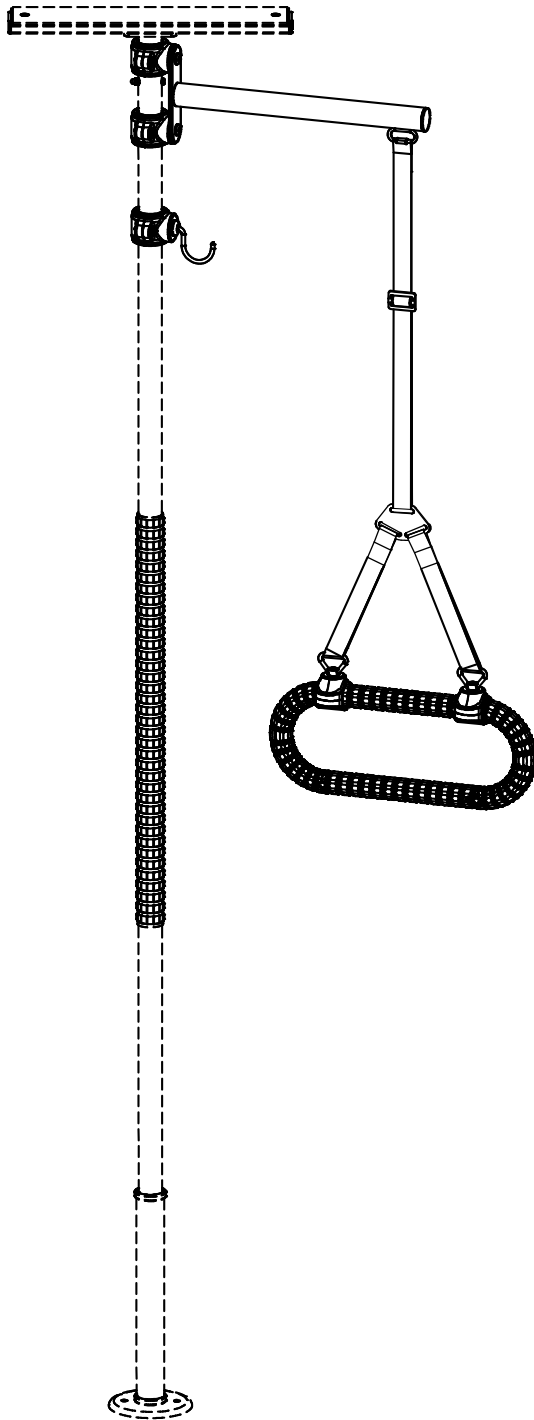


Table of Contents


- SG602 Installation Instruction (4 Pages)

Specifications:


- Product Name: SafetyGrip® TRAPEZE
- Product#: SG602
- Maximum User Weight: 350lbs (158kg)

KIT CONTENTS

For regular installations you will need a Step Ladder, Safety Glasses and other tools required are included in the Kit.

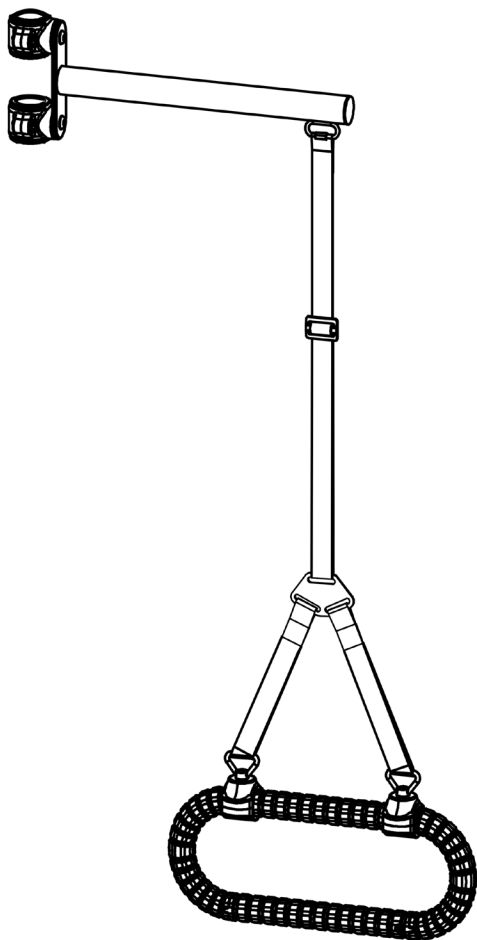

STORAGE HOOK

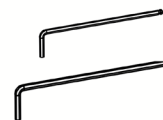

HEX KEYS

PRODUCT INFORMATION

This Titan SG602 SafetyGrip® Trapeze feature our patented interlocking SafetyGrip® and the long-lasting protection of embedded antimicrobial AlphaSan® RC5000 silver ions powder for all parts except hardware. For more details about AlphaSan®: <https://www.milliken.com/en-us/businesses/chemical/product/alphasan>

- The Titan SG602 SafetyGrip® Trapeze is designed to be used with these Titan Series Poles - SG600; SG610; SG615 & SG650.
- The SG602 is designed to assist with sit-up, reposition, balance, stand, or transfer for persons weighing up to 350lbs (158kg) and is **not designed** to bear the full weight of the user.
- The Trapeze strap is adjustable to allow you to adjust the height of the Trapeze to where you want it.
- The Trapeze can be stored on the Storage Hook when you are not using it.


INSTALLATION

STEP 1: Open the Trapeze Arm Clamps

First put on Safety Glasses. Open the clamps by removing the Bolts and Nuts from the clamps using the large Hex Key.

STEP 2: Install the Trapeze Arm on the Pole

Install the Trapeze Arm onto the top of your Grab Pole, where you want it, re-insert the Bolts and Nuts and tighten securely using the large hex key. Securely tighten the cone point security screws in the two Bolts using the small hex key.

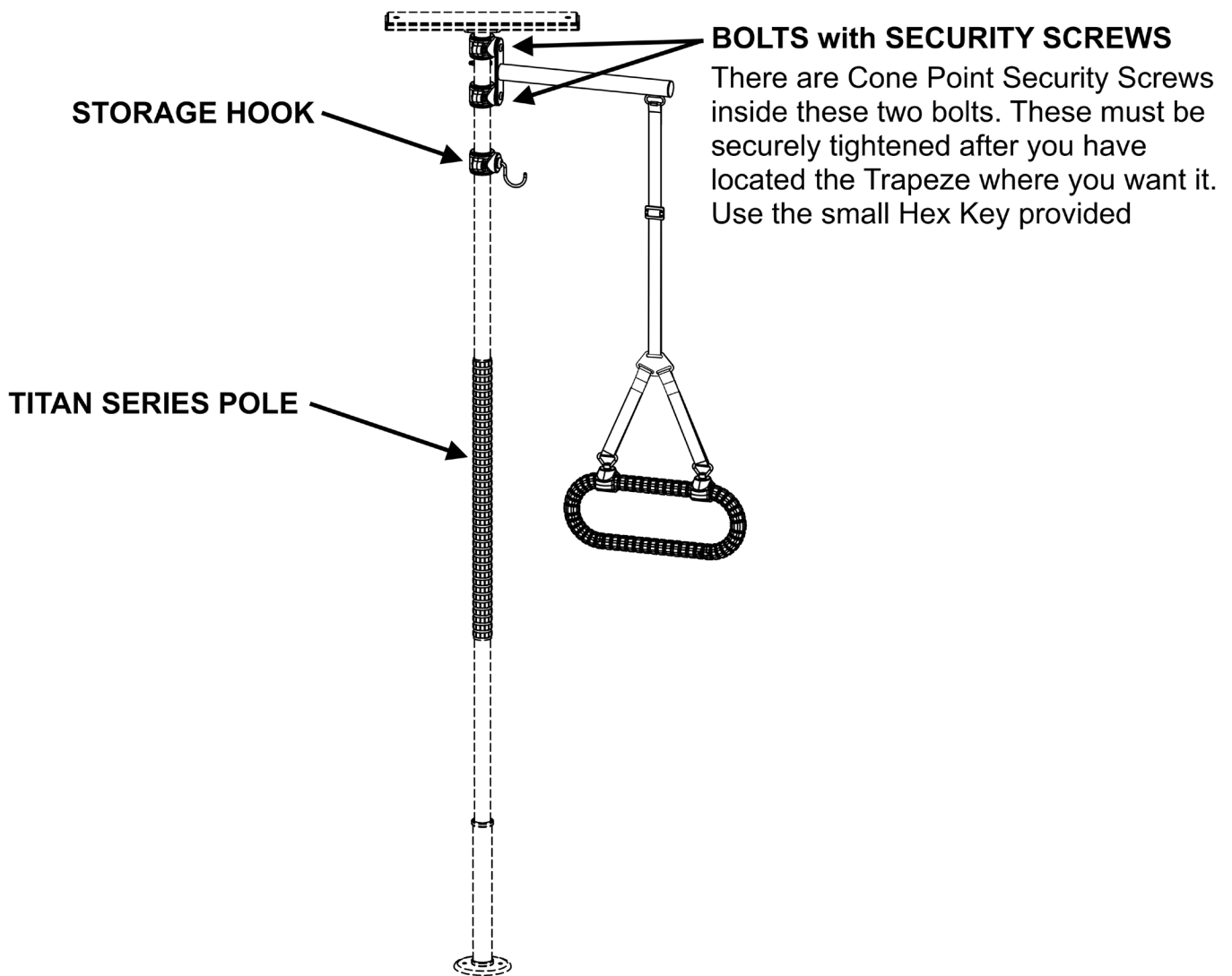

STEP 1

STEP 3: Decide how high you want the Safety Grip Trapeze

Decide how high you want the Safety Grip Trapeze above the bed and adjust the straps accordingly.

MAINTENANCE

- **VERY IMPORTANT:** It is important to inspect the Product(s) frequently.
- Stop using immediately if damaged or broken, or if parts are missing.
- Stop using immediately if Product(s) shift out of the original installed position(s) until they are readjusted into the correct position.
- To clean your Trapeze, use a damp cloth with soap and water or mild household cleaner, wipe clean and dry before using.

Warranty

For Warranty details, refer to the Warranty details on the Hartmobility Inc.

Website: www.hartmobility.com/Warranty or call 1-800-998-0774 ext. 3



Questions. Comments. Concerns.

Contact Customer Service today.

info@SafetyGrip.com

www.SafetyGrip.com

www.Hartmobility.com

1(800) 998-0774

Hartmobility Inc.
424 Williamson-Zebulon Rd.
Williamson, GA 30292

Hartmobility reserves the right to update and revise our products. Refer to our website for the most up-to-date version of this document.

Copyright © 2024 Hartmobility Inc. All Rights Reserved.

