


Hartmobility Easy Anchor™ Installation Instructions

Hartmobility Easy Anchor™ KIT for Drywall and Hollow Wall Mounting

Models: EA101

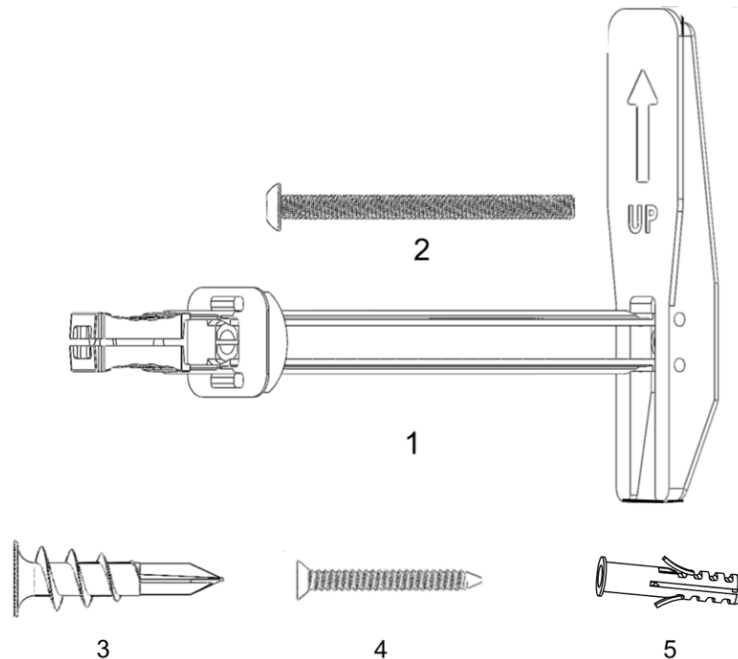

1. Hartmobility Easy Anchor™ Fastener Kit x 1
2. BHC Screw 3/16" x 2-3/8" x1
3. Self Drilling Drywall Anchor x1
4. Screw x1
5. Plastic Wall Plug x1

Tools Required:

Safety Glasses	Masking tape	1 inch hole saw for either Drywall or Tiles	
Tape Measure	Spirit Level	If you wall is Tiled you will need a 1/4" drill bit for Tiles	
Pencil	Electric Drill	Caulking	Phillips Screw Driver

Technical Data:

- Our Hartmobility Easy Anchor™ Fastener Kit is designed to allow items to be safely mounted onto hollow structural walls such as drywall, drywall with tile, cement board, fiberglass, onto 3/4 inch thick plywood blocking and into hollow walls.
- Items can be Grab Bars and Accessories, Shelves, TVs, Screens, Picture Frames, Exercise Equipment etc.
- Designed to make a Grab Bar ADA Compliant when mounted with mounting feet on Drywall and hollow wall or mounting feet on drywall and hollow wall and on stud.
- ADA Compliant, including ADA 609.8 Structural Strength.
- Will support up to 500lbs when installed correctly on structurally sound walls.
- Allows Grab Bars and Accessories to be mounted anywhere on walls where you want them.
- Easy Installation.



INSTALLATION

STEP1: Prepare and protect your wall:

Find where you want to mount your Grab Bar or Accessory. Put masking tape in this area.

STEP 2: Mark the holes.

Hold the Grab Bar or Accessory in place and mark the top and bottom hole in the mounting plate.

STEP 3: For Drywall.

In the top hole drill a 1" hole. In the bottom hole install the self drilling drywall anchor.

STEP 3: For Tiled Walls.

In the top hole drill a 1" hole and a 1/4" hole in the bottom hole. Install the Plastic Wall Plugs.


STEP 4: Install the 'Easy Anchor™.


Push the 'Easy Anchor™ through the 1 inch hole with the plastic rails and make sure that the 'Easy Anchor™ is pointing up as the arrow on it shows. Pull the 'Easy Anchor™ tight against the back surface of the wall and push the plastic retainer ring tight against the front surface of the wall. Snap or cut off the plastic rails.

STEP 5: Mount the Grab Bar or Accessory.


First, to make the connection water tight we suggest that you put a small bead of caulking on the mounting plates. Insert the long screw through the top hole in each mounting plate and tighten firmly. Put the drywall anchor screw through the bottom hole of the mounting plate and tighten. Snap on the plastic mounting feet covers. Your installation is complete.

See the diagrams that follow:

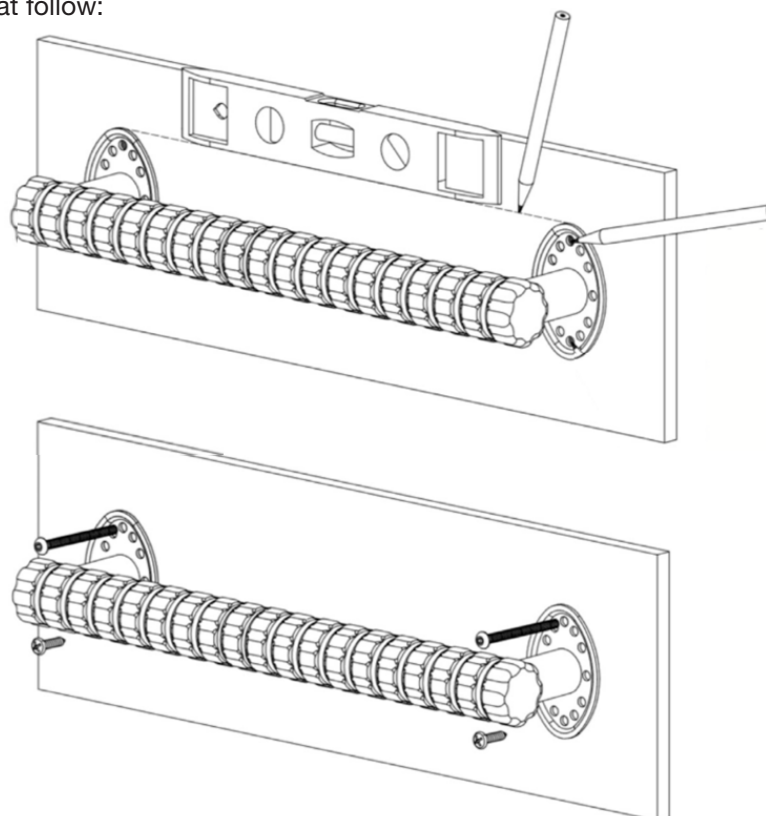

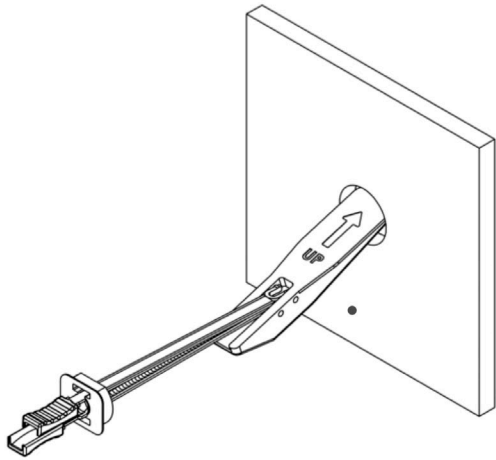

Drill 1 inch hole and push the 'Easy Anchor™ through

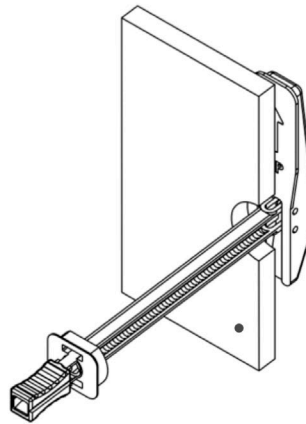

Pull the 'Easy Anchor™ tight against wall




Slide the retainer ring tight against the surface. Snap or cut the rails.

Warranty

For Warranty details, refer to the Warranty details on the Hartmobility Inc.

Website: www.hartmobility.com/Warranty or call 1-800-998-0774 ext. 3

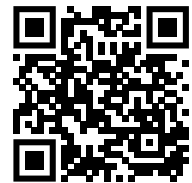

Questions. Comments. Concerns.

Contact Customer Service today.

info@SafetyGrip.com

www.SafetyGrip.com

www.Hartmobility.com

1(800) 998-0774

Hartmobility Inc.
424 Williamson-Zebulon Rd.
Williamson, GA 30292

Hartmobility reserves the right to update and revise our products. Refer to our website for the most up-to-date version of this document.
Copyright © 2025 Hartmobility Inc. All Rights Reserved.

